




# T-EZ Pro® Cleaning Indicator

## Assessment Aid

### ILLUSTRATION OF PASS/FAIL

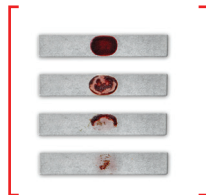

FAIL

### FAIL

- Spot of test soil completely visible
- Spot of test soil visible
- Spot of test soil visible
- Spot of test soil slightly visible

### ACTION

If failed with instruments DO NOT release instruments.

Check mechanical washer's:

- temperature
- dosage
- time

In case of repeated failure (with or without instruments):

- address policy
- call service technician



PASS

### PASS

- Spot of test soil completely removed


### ACTION

Release batch. No further action needed.

For more information and additional languages,  
please see <https://biologicalindicators.mesalabs.com/cleaning-indicators>

# T-EZ Pro® Reinigungsindikator

## Bewertungstabelle

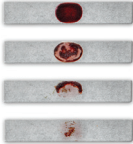Ergebnis-Darstellung	Fehlerhaft	Maßnahme
	<ul style="list-style-type: none"><li>• Referenzindikator: Prüfanschmutzung vollständig sichtbar</li><li>• Prüfanschmutzung sichtbar</li><li>• Prüfanschmutzung sichtbar</li><li>• Prüfanschmutzung leicht sichtbar</li></ul>	<p>Bei Beladung mit Instrumenten: Instrumente NICHT freigeben. Überprüfen der Parameter:</p> <ul style="list-style-type: none"><li>• Temperatur</li><li>• Dosierung</li><li>• Zeit</li></ul> <p>Bei wiederholtem Ausfall (mit oder ohne Instrumente):</p> <ul style="list-style-type: none"><li>• Servicetechniker anrufen</li></ul>
	<p><b>Erfolgreich</b></p> <ul style="list-style-type: none"><li>• Prüfanschmutzung <u>vollständig</u> entfernt</li></ul>	<p><b>Maßnahme</b></p> <p>Charge freigeben. Keine weiteren Maßnahmen erforderlich.</p>